

HOW TO USE THE TOUCH + NO TOUCH THERMOMETERS



1. BEFORE TAKING A MEASUREMENT, REMOVE DIRT OR HAIR FROM THE FOREHEAD AREA



2. REMOVE PROTECTIVE CAP



3. PRESS THE ON/OFF BUTTON



4. IF USING A TOUCH METHOD, POSITION THE THERMOMETER IN FRONT OF THE FOREHEAD IN THE MIDDLE OF THE EYEBROWS TOUCHING THE FOREHEAD.



5. IF USING NO TOUCH METHOD, POSITION UP TO 5CM AWAY USING THE PROXIMITY SENSOR AND AIMING LIGHT. FOR PATIENTS MEASURING THEIR OWN TEMPERATURE, IT IS RECOMMENDED TO USE THE "TOUCH" OPTION INSTEAD OF "NO TOUCH" AS IT IS MORE CONVENIENT.



6. KEEP THE PATIENT'S HEAD STILL DURING MEASUREMENT. DO NOT MOVE THE THERMOMETER DURING MEASUREMENT.



7. PRESS THE TEMPERATURE BUTTON TO TAKE A MEASUREMENT